

Connect your Accu-Chek meter to the mySugr® app and turn numbers into better outcomes¹


The mySugr app automatically and wirelessly captures your readings so you get the full picture of what's affecting your blood glucose.
No more paper logs.

Download the app here


It's easy. Scan the code with your phone to download, then use your email to create an account.




How to connect your Accu-Chek meter to the my Sugr app - instructions for ANDROID

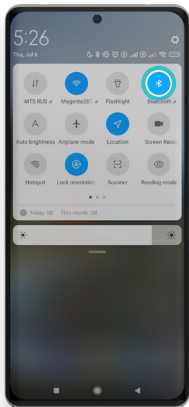


1. Make sure Bluetooth is turned on.

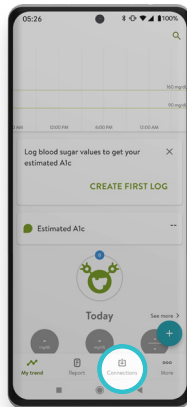


2. In the mySugr app, tap on "Connections" at the bottom of the screen.

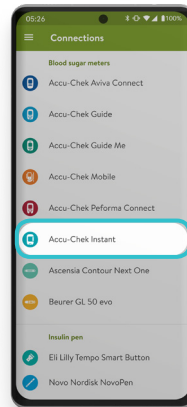


3. Choose your device from the list.

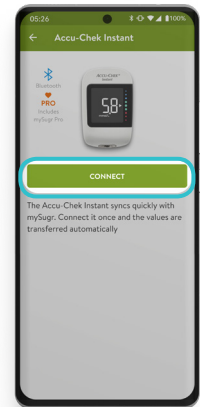


4. Tap the Connect button to start the pairing.

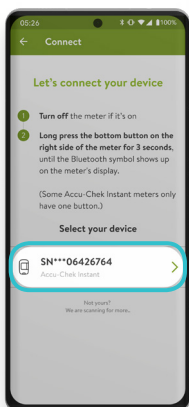


5. Follow the on-screen instructions.

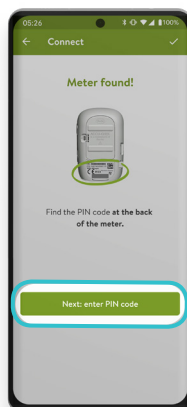

6. Find the PIN code displayed on your meter's screen.

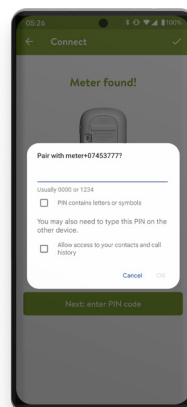

7. When prompted, enter the PIN code to connect your device.

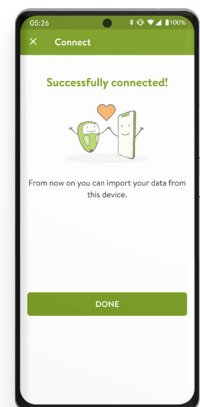

8. Success! Your device is now connected to the mySugr app.


Watch our **Android** video instruction to connect quickly




How to connect your Accu-Chek meter to the my Suger app - instructions for iOS




1. Make sure Bluetooth is turned on.

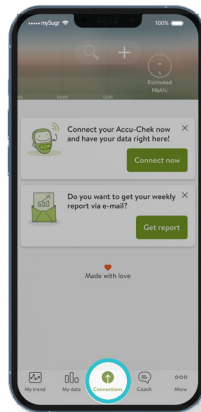


2. In the mySugr app, tap on "Connections" at the bottom of the screen.

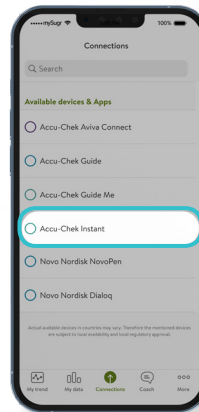


3. Choose your device from the list.

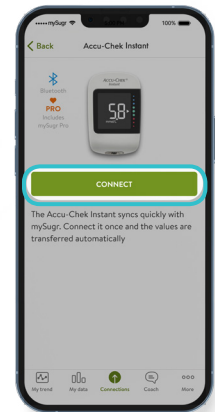


4. Tap the Connect button to start the pairing.

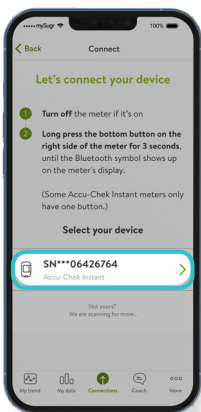


5. Follow the on-screen instructions.

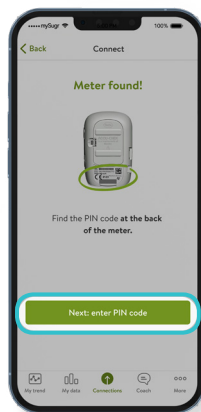

6. Find the PIN code displayed on your meter's screen.

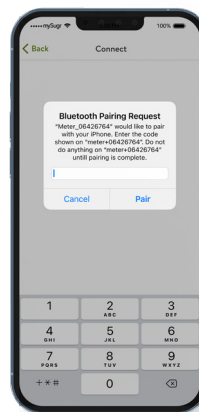

7. When prompted, enter the PIN code to connect your device.

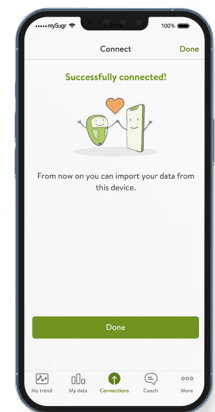

8. Success! Your device is now connected to the mySugr app.

Need help? info@accu-chek.co.za

Watch our **iOS** video instruction to connect quickly



Reference: 1. Dehong F, Mayer H, and Kober J. Real-World Assessments of mySugr Mobile Health App. Diabetes Technol Ther. 2019;21(S2):S235-S240.

*To check if you have a mobile device that is compatible with the mySugr app, contact our customer support centre.

BE PART OF OUR ONLINE COMMUNITY

Join the conversation at RocheDiabetesCareSSA | Visit the Roche Diabetes Care website at www.rochediabetescaremea.com
Email: info@accu-chek.co.za | Toll Free: 080-34-22-38-37 (South Africa only)

ACCU-CHEK, ACCU-CHEK INSTANT and MYSUGR are trademarks of Roche. All other product names and trademarks are property of their respective owners.
© 2024 Roche Diabetes Care | Roche Diabetes Care South Africa (Pty) Ltd. | Hertford Office Park, Building E | No 90 Bekker Road | Midrand, 1686, South Africa | ZA-461

