EN FR DE ES PT UK ID MS TH AR

ACCU-CHEK® Safe-T-Pro Uno

05888662, 06372481

EN Sterile, single-use lancening device

Read these instructions for use before you obtain blood with this sterile, single-use lancening device.

These instructions for use feature the following 2 symbols:

A This symbol indicates a **possible risk of injury or of damage to your health**.

I This symbol draws your attention to **important information**.

Intended Use

The sterile, single-use lancening device is intended for obtaining capillary blood from the fingertip or, if the patient is a child under 1 year, from the heel.

Components of the single-use lancening device (see illustration 1)

- 1 Description
- 2 Release button
- 3 Housing
- 4 Sterility cap of the needle

The special cut and small diameter of the needle (Ø 0,36 mm, 28 G) allow you to use this lancening device to obtain blood from your patients in a way that is virtually pain free.

The sterility cap keeps the needle **sterile**.

Preparing to obtain blood

2 Children younger than 1 year

With children younger than 1 year of age, the capillary blood must be obtained from the heel. Obtaining blood must occur from the lateral areas of the sole (see black areas in illustration 2).

The puncture must occur vertically to the sole (see arrows in the enlargement). While obtaining blood, the child should be held with its foot vertically facing the floor. A site which has already been punctured must not be punctured a second time.

To improve blood flow at the heel before obtaining blood, you can carefully warm the heel, for example by wrapping it in a warm, moist cloth of a maximum 42 °C.

Wash the child's heel with warm water and soap and rinse thoroughly. Disinfect the heel with an alcohol wipe. Afterwards, let the heel dry completely.

3 Children as of 1 year and adults

We recommend obtaining capillary blood from the side of the fingertip, as this puncture site registers the least amount of pain. Any finger can be used for the puncture (see black areas in illustration 3).

Wash the patient's hands with warm water and soap and rinse thoroughly. Afterwards, let the hands dry completely. This ensures that the puncture site is clean and increases blood flow. If you are using alcohol wipes in addition, the finger must also be allowed to dry completely.

Obtaining blood

4 Removing the sterility cap

- Twist the sterility cap one quarter turn and remove it (see illustration 4).

A Do not use the single-use lancening device if the sterility cap has been previously removed.

5 + 6 Obtaining blood

- Hold the single-use lancening device between the index finger, middle finger and thumb (see illustration 5).
- Press the single-use lancening device firmly against the chosen puncture site.
- Using your thumb, press the release button down completely (see illustration 6).
- Remove the single-use lancening device from the puncture site.

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- Wait for a blood drop to emerge from the puncture site.
- Gently massage the finger towards the fingertip or exert slight pressure on the heel to encourage a blood drop to form.

A For some diagnostic tests, the fingertip/heel must not be vigorously massaged or compressed, as doing so may produce incorrect test results.

I Always discard used single-use lancening devices in accordance with your facility's disposal procedures.

Last update

2019-04

Customer Support and Service Centre

Roche Diabetes Care Australia Pty. Limited

2 Julius Avenue, North Ryde

NSW 2113, Australia

ABN 69 602 140 278

Accu-Chek Enquiry Line: 1800 251 816

www.accu-chek.com.au

Canada

Accu-Chek Customer Care: 1-800-363-7949

www.accu-chek.ca

India

Accu-Chek Customer Care Helpline:

1800 120 6020 (Toll-free)

(Monday to Saturday, 7:30 AM to 9:30 PM)

SMS "HELP" to 5757535*

*Charges applicable as per your network provider

Email: india.accu-chek@roche.com

www.accu-chek.in

Indonesia

Customer Support and Service Centre

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