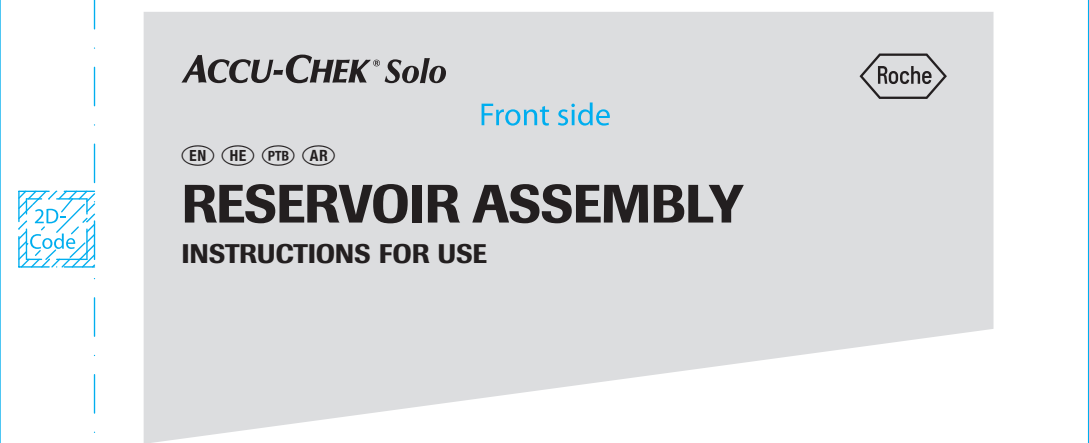
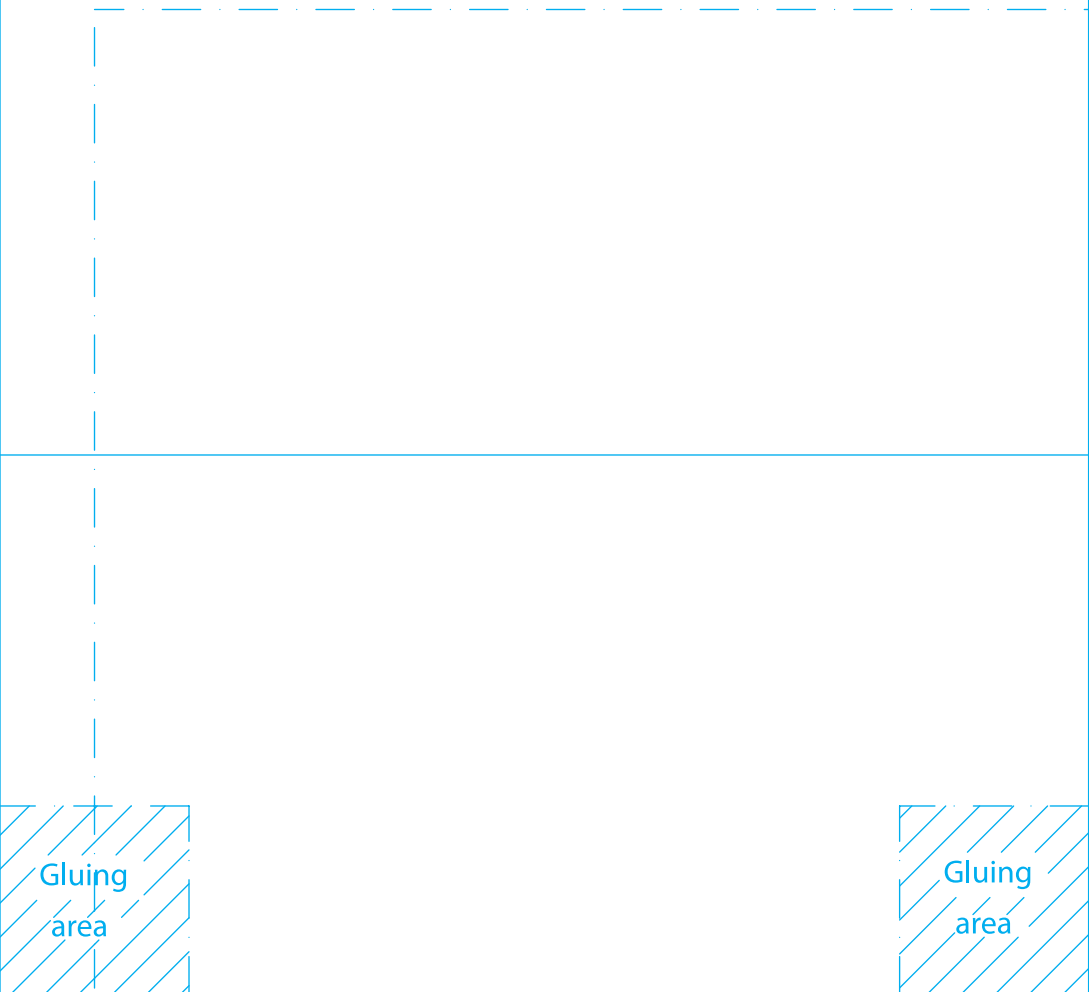
🇬🇧 Instructions for use
Accu-Chek Solo Reservoir Assembly
Intended Use
The Accu-Chek Solo reservoir assembly (including the filling aid and the handle for the piston rod) is a part of the micropump in addition to the Accu-Chek Solo pump base. The reservoir is a sterile container for holding the insulin that the micropump delivers to the body.
Package Contents and Storage Conditions
See outer packaging. You can find the symbol explanations in the User's Manual of the Accu-Chek Solo micropump system.
Component Overview
A Reservoir
B Filling aid (blue)
C Handle for the piston rod (blue)
D Protective film for battery (orange)
E Reservoir needle
F Piston rod
Take care not to touch the reservoir needle (E). You might injure yourself.
This product contains sharp parts which can cause injury. Keep pointed or sharp-edged parts away from small children and people who could be injured by them.

⚠️ WARNING
Risk of suffocation
This product contains small parts that can be swallowed.
Keep the small parts away from small children and people who might swallow small parts.

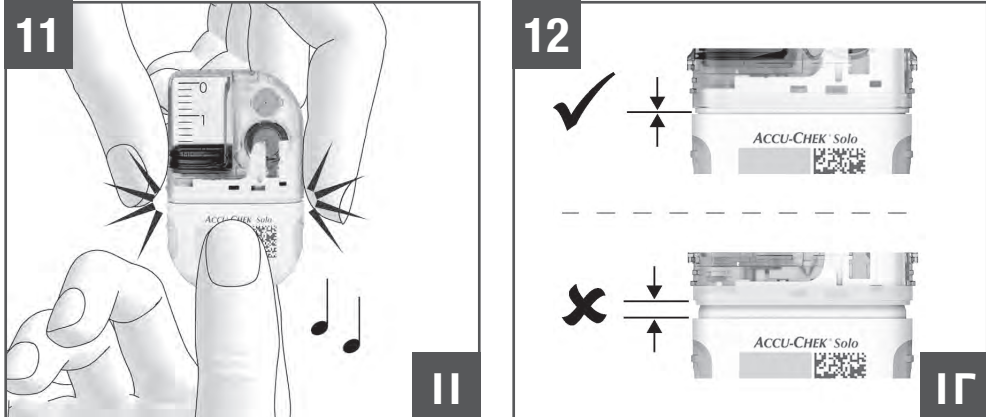
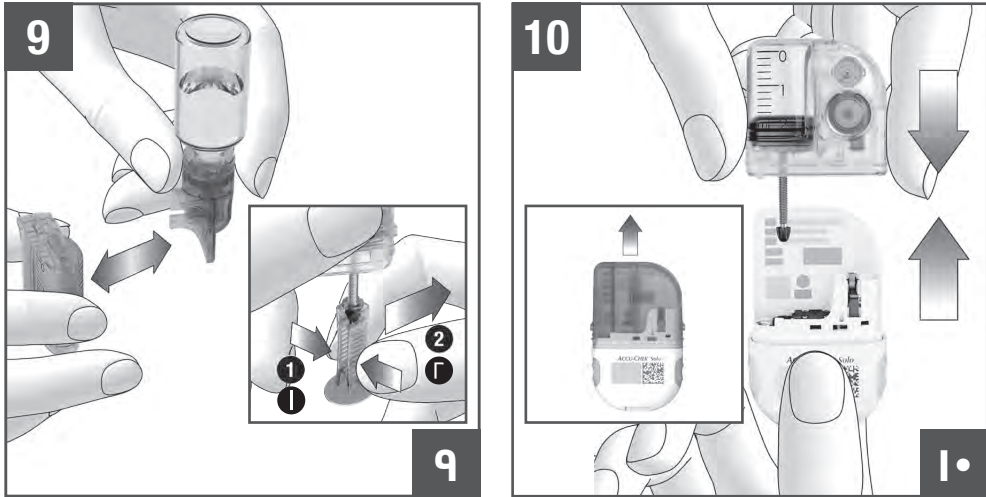
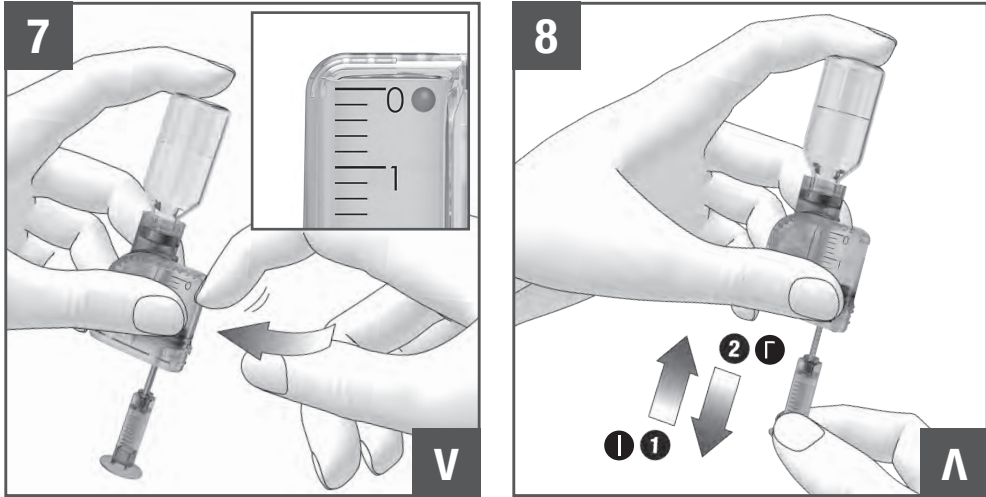
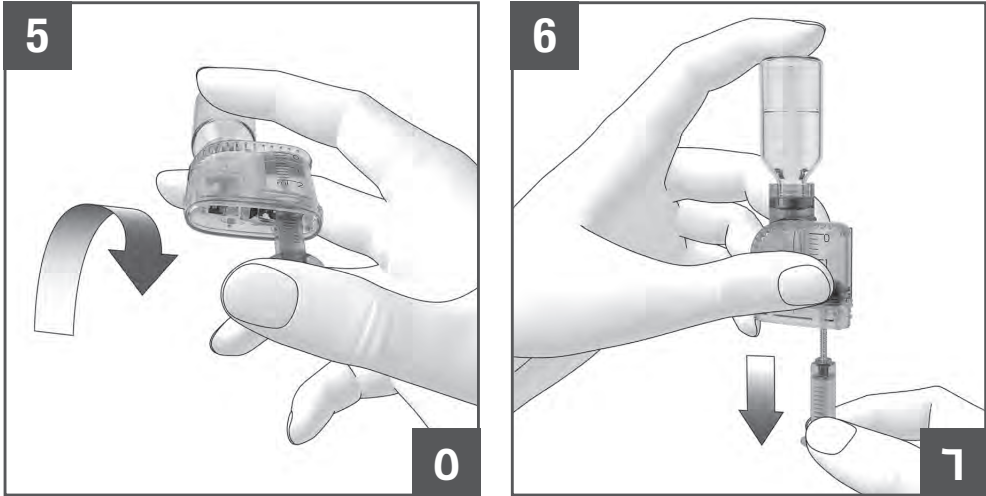
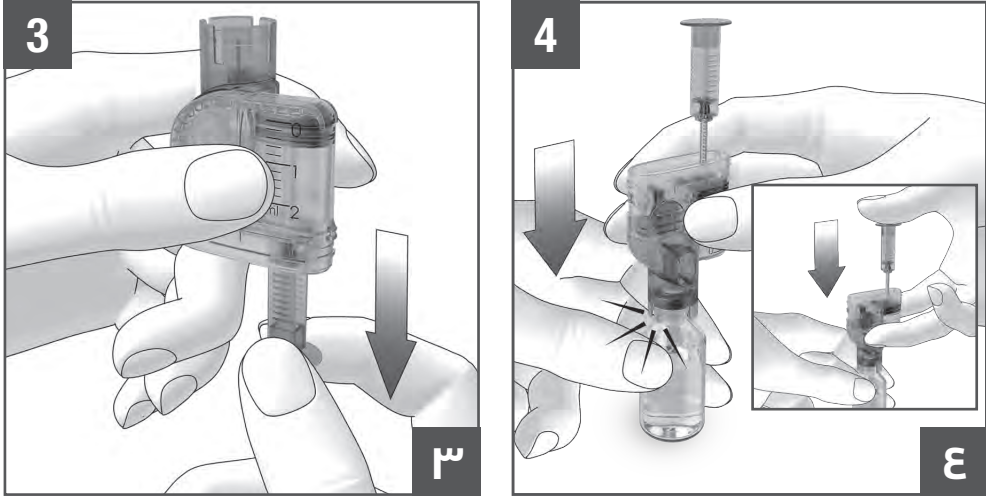
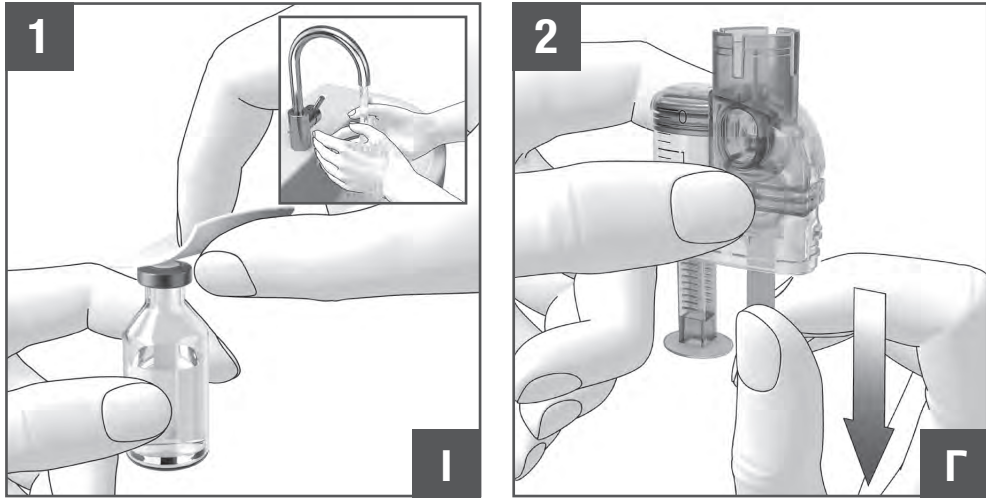
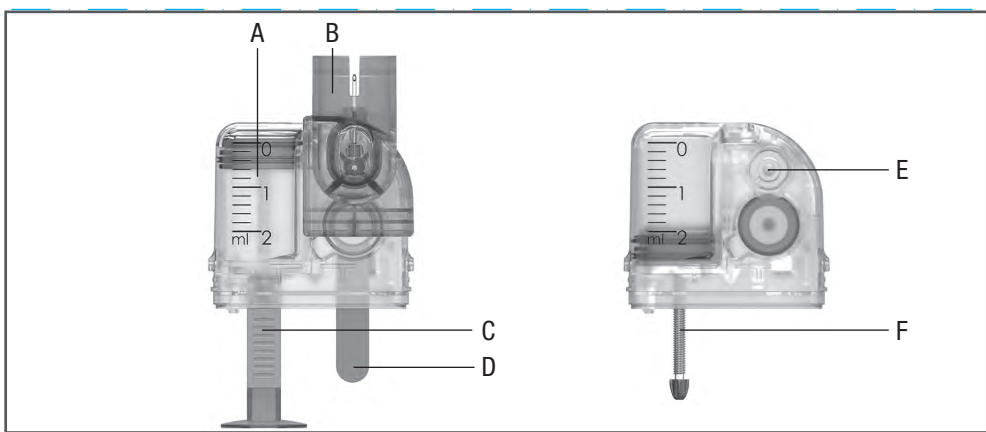
Before You Get Started
Read these instructions for use and the instructions in the User's Manual of the Accu-Chek Solo micropump system. Take particular note of all warnings, information, and notes required for the safe and proper use of the micropump system.
This is a sterile, single-use product that has been sterilized with ethylene oxide. You can use the reservoir assembly for up to 4 days.
Use and store the insulin in compliance with the manufacturer's specifications and pay attention to the use by date. For more information on compatible insulins, see the User's Manual of the Accu-Chek Solo micropump system.

Filling the Reservoir with Insulin
⚠️ WARNING
Risk of hypoglycemia (low blood glucose level) and hyperglycemia (high blood glucose level)
• The micropump system is intended for use with U100 insulin. Only use the micropump to deliver rapid-acting U100 insulin.
• Make sure that the insulin is at room temperature before you fill the reservoir. If insulin is used which is not at room temperature, the insulin may expand in volume. This can lead to inaccuracy in insulin delivery.
The reservoir must be filled with at least 80 U. The reservoir has a maximum holding capacity of 200 U (2.0 ml).
In addition to the reservoir assembly, you need an insulin vial with U100 insulin and a form of disinfectant, such as a sterile alcohol wipe.
Visually inspect this component before using it with the Accu-Chek Solo micropump system. If you notice anything unusual, use a new component.

To replace a component of the micropump system, always use the *Replace* menu on the diabetes manager. To view an animated video of the handling steps, tap *Help* on the diabetes manager.
❶ Wash your hands and dry them thoroughly.
Disinfect the top of the insulin vial with a sterile alcohol wipe. Allow the disinfected top of the insulin vial to dry.
❷ Slowly and carefully pull out the battery's protective film (D) downwards in the direction of the arrow to activate the battery.
❸ Hold the round part of the handle (C) firmly and pull it downwards in the direction of the arrow in order to fill the reservoir (A) with air. Fill the reservoir with the volume of air that you later want to fill with insulin.
❹ Place the insulin vial on a flat and solid surface (for example, a table top) and hold it firmly.
Place the filling aid (B) onto the vial. Push the filling aid downwards until you hear it click into place. Press the handle all the way down in the direction of the arrow to fill the insulin vial with air.
❺ Turn the reservoir assembly together with the insulin vial upside down so that the vial is above the reservoir.
❻ Slowly pull the handle downwards in the direction of the arrow to fill the reservoir with insulin. Try to ensure that as few air bubbles as possible form in the reservoir.
❼ Check the reservoir from several angles to make sure that there are as few air bubbles as possible in the reservoir. To release air bubbles from the reservoir, hold the reservoir at an angle. Gently flick your finger against the reservoir several times.

⚠️ **WARNING**
Risk of serious harm
Air bubbles can cause incorrect insulin delivery. Air bubbles may be present in the insulin or adhere to the inner surfaces of the reservoir. Air bubbles equal or larger than the air bubble indicated in the illustration, must be removed.
If it is not possible to remove these air bubbles you must replace the reservoir.
❶ Slowly push the handle upwards in the direction of the arrow ❶ to remove air bubbles from the reservoir. Slowly pull the handle downwards in the direction of the arrow ❷ until the reservoir is filled with the required amount of insulin again.
❷ Remove the filling aid from the reservoir by pulling it off laterally.
Compress the handle on the ribbed surface ❸ and then remove the handle laterally from the piston rod (F) ❹.
❸ Remove the blue reservoir cap from the pump base. Push the filled reservoir onto the pump base until both parts are tightly connected.
❹ When the reservoir and the pump base are connected to each other correctly and the battery is activated, the micropump issues the *Start* sound. If this sound is not issued, check whether the battery is activated and repeat step 10.
❺ Make sure that there is no gap between the reservoir and the pump base.
The pump base and the reservoir are only connected correctly when the *Start* sound is issued and when the pump base and the reservoir are seamlessly connected.

Disposal
Dispose of used products according to local regulations. For information about correct disposal, contact your local council or authority. Dispose of the reservoir assembly and the filling aid safely, in such a way that no third party can injure themselves.
In US, distributed by:
Roche Diabetes Care, Inc.
Indianapolis, IN 46256, USA
Toll Free: 1-800-688-4578
accu-check.com

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